

Medical Surface Disinfectant Wipes

Cleans, Disinfects and Deodorizes non-porous surfaces in One Minute

99.99% EFFECTIVE IN KILLING COMMON BACTERIA



MinutePure is a fast and effective disposable wipe that KILLS BACTERIA, FUNGI, AND VIRUSES* IN ONE MINUTE. The pre-saturated wipe, with its antibacterial formula, cleans and disinfects in one step, saving time. The all-purpose wipe cleans and cuts grease, cleans tough stains, cuts through tough grime, eliminates odors, and leaves your non-porous surfaces fresh and clean.



Available in 80 and 160 count canisters.

FEATURES AND BENEFITS

- **CONVENIENT:** One-step cleaning. Saves time. Antibacterial, disinfecting and scrubbing power all in one. Cleans and Disinfects in just 1 minute.
- **DISINFECTANT:** BACTERICIDAL / TUBERCULOCIDAL / FUNGICIDAL / VIRUCIDAL*
- **BACTERICIDAL:** MinutePure is an effective disinfectant on hard non-porous surfaces, in the presence of an organic load, against Escherichia coli O157:H7, Pseudomonas aeruginosa, Salmonella enterica, Staphylococcus aureus, Streptococcus pyogenes, Vancomycin resistant Enterococcus faecium, Methicillin resistant Staphylococcus aureus, Klebsiella pneumonia, Listeria monocytogenes, Acinetobacter baumannii, Campylobacter jejuni, and Burkholderia cepacia, when the treated surface is allowed to remain wet for 1 minute. Allow surfaces to air dry.
- **TUBERCULOCIDAL:** MinutePure is an effective disinfectant on hard non-porous surfaces, in the presence of an organic load, against Mycobacterium bovis, when the treated surface is allowed to remain wet for 1 minute. Allow surfaces to air dry.
- ***VIRUCIDAL:** MinutePure is an effective virucide on hard, non-porous surfaces, in the presence of an organic load, against Poliovirus serotype 1, Hepatitis B Virus, Hepatitis C Virus, Rhinovirus, Norovirus, Human Coronavirus, Rotavirus Strain WA, Respiratory Syncytial Virus, Human Immunodeficiency Virus, Herpes Simplex Virus Type 1, and Influenza Virus type A2, when the treated surface is allowed to remain wet for 1 minute. Allow surfaces to air dry.
- **DEODORIZER:** Eliminates odors. Unscented. Fragrance Free.
- **MANUFACTURER:** Made in the USA

Contact your Vital Pure representative today: Paul Cohen (914) 649-9793

ACTIVE INGREDIENTS

Ethyl Alcohol.....	72.50%
Dedecyl Dimethyl Ammonium Chloride.....	0.33%
OTHER INGREDIENTS*.....	27.17%
TOTAL.....	100.00%

*Other ingredients do not include the weight of the dry wipes

FOR USE ON

MinutePure is formulated to disinfect hard, non-porous, inanimate environmental surfaces such as floors, walls, metal surfaces, stainless steel surfaces, glazed porcelain, glazed ceramic tile, plastic surfaces, bathrooms, shower stalls, bathtubs, cabinets, patient care equipment and point-of-care equipment.

FOR USE IN

Hospitals and healthcare settings, including: Ambulances, Anesthesia, CAT Labs, CCU, Central Supply, Clinics, Dental Offices, Dialysis Clinics, Doctor's Offices, Emergency Rooms, Food Service and Food Prep Areas, Home Health Care, Hospices, Laboratory, Laundry Rooms, Long Term Care Facilities, Newborn Nursery, Nursing Homes, Offices, Operating rooms, Pediatrics, Physical Therapy, Physician's Offices, Ophthalmic Offices, Radiology, Recovery Room, Rehabilitation, Respiratory Therapy, Ultrasound, Transport Vehicles, Waiting Areas, X-Ray. Critical Care Areas: CCU, ER, ICU, NICU, PICU, Surgery, Surgical Prep, and Surgical Recovery Units.

DISINFECTING

To disinfect hard, non-porous surfaces, use one or more wipes as necessary, to thoroughly wet the surface to be treated. Treated surface must remain visibly wet for one minute to achieve complete disinfection of all pathogens listed on this label. For visibly soiled surfaces, clean before following disinfecting instructions. Allow surfaces to air dry. For surfaces that may come in contact with food, a potable water rinse is required.

TECHNICAL SPECS

160 Count Canister

- 12 cases of 6 per row
- 10 rows per pallet
- 1836 lbs. per pallet
- 720 tubs per pallet
- 24 pallets per truck
- 17,280 tubs per truck

80 Count Canister

- 24 cases of 6 per row
- 10 rows per pallet
- 1865 lbs. per pallet
- 1440 tubs per pallet
- 24 pallets per truck
- 34,560 tubs per truck

Packaging: Canister

Color: White

Type: Pre-Moistened

Wipe Size: 6" x 6.7" (15cm x 17cm)

Units: 160 sheets/roll

Quantity/Case: 6 rolls/case (80/160 ct)

DIRECTIONS FOR USE: It is a violation of Federal Law to use this product in a manner inconsistent with its labeling. This product is not for use on critical and semi-critical medical device surfaces.

CLEANING: Use one or more wipes, as necessary, to wet surface sufficiently and to thoroughly clean the surface.

DISINFECTING: To disinfect hard, non-porous surfaces, use one or more wipes, as necessary, to thoroughly wet the surface to be treated. Treated surface must remain visibly wet for one (1) minute to achieve complete disinfection (or kill) of all pathogens listed on this label. For visibly soiled surfaces, clean before following disinfecting instructions. Allow surfaces to air dry. For surfaces that may come in contact with food, a potable water rinse is required.

To Dispense Wipes: Open lid, remove entire cover, remove and discard seal. From center of wipes roll pull up a corner of the wipe, twist it and thread it through the hole in the container cover. Pull through about one inch. Replace cover tightly. Pull out first wipe and tear off at a 45° angle. Remaining wipes feed automatically, ready for next use. When not in use, keep the tip of the exposed wipe inside the well in the cover, and the lid strapped closed.

EFFICACY TESTS HAVE DEMONSTRATED THAT MINUTEPURE IS AN EFFECTIVE, VIRUCIDE, BACTERICIDE, TUBERCULOCIDE AND FUNGICIDE IN THE PRESENCE OF ORGANIC SOIL.

MinutePure will not harm most surfaces: including stainless steel, aluminum, plastics, fiberglass, vinyl, and most painted surfaces. For plastic and painted surfaces, spot test on an inconspicuous area before use. Do not use on unpainted wood. Not for use on people or pets.

PRECAUTIONARY STATEMENTS
Hazardous to Humans and Domestic Animals:

CAUTION: Causes moderate eye irritation. Avoid contact with eyes or clothing. Wash thoroughly with soap and water after handling and before eating, drinking, chewing gum, using tobacco, or using the toilet.

FIRST AID
IF IN EYES: Hold eyes open and rinse slowly and gently with water for 15–20 minutes. Remove contact lenses if present, after the first 5 minutes, then continue rinsing. Call a poison control center or doctor for treatment advice.
Have the product container or label with you when calling a poison control center, doctor, or going for treatment. For emergency information concerning this product, call the National Pesticides Information Center (NPIC) at 1-800-858-7378 seven days a week, 6:30 am to 4:30 pm Pacific Time (NPIC Web site: www.npic.orst.edu).

PHYSICAL OR CHEMICAL HAZARDS:
Flammable. Keep away from heat or open flame.

STORAGE AND DISPOSAL
Do not contaminate water, food or feed by storage or disposal.
PESTICIDE STORAGE: Always store this product and all pesticides in the original container. If a leaky container must be contained within another, mark the outer container to identify the contents. Protect from extreme heat or cold.
PESTICIDE WIPES DISPOSAL: Waste resulting from the use of this product must be disposed of on site or at an approved waste disposal facility.
CONTAINER DISPOSAL: Nonrefillable container. Do not reuse or refill this container. Offer for recycling if available, or discard empty container in trash.

KILLS BACTERIA / FUNGI / VIRUSES* IN ONE MINUTE

99.99% EFFECTIVE IN KILLING COMMON BACTERIA

UNSCENTED | FRAGRANCE FREE | BLEACH-FREE

ACTIVE INGREDIENTS:	
Ethyl Alcohol	72.50%
Didecyl Dimethyl Ammonium Chloride	33%
OTHER INGREDIENTS*	27.17%
Total	100.00%

*Other ingredients do not include the weight of the dry wipes.

KEEP OUT OF REACH OF CHILDREN
CAUTION: See side panels for additional precautionary statements.

NET CONTENTS: 1lb. 8 oz.
160 Pre-Moistened Wipes (6 in. by 6.7 in.) (15cm x 17cm)

MADE IN THE U.S.A.

***MinutePure kills the following pathogens on hard, non-porous surfaces in 1 minute when the treated surface is allowed to remain wet for 1 minute. Allow surfaces to air dry.**

Pathogen	ATCC Number	Organic Load	Kill Time
Acinetobacter baumannii	19606	5%	1 minute
Burkholderia cepacia	25416	5%	1 minute
Campylobacter jejuni	29429	5%	1 minute
Escherichia coli O157:H7 [E. coli]	35150	5%	1 minute
Klebsiella pneumoniae	4352	5%	1 minute
Listeria monocytogenes	984	5%	1 minute
Methicillin resistant Staphylococcus aureus (MRSA)	35591	5%	1 minute
Pseudomonas aeruginosa	15442	5%	1 minute
Salmonella enterica	10708	5%	1 minute
Staphylococcus aureus [Staph]	6538	5%	1 minute
Streptococcus pyogenes [Strep]	19615	5%	1 minute
Vancomycin resistant Enterococcus faecium [VRE]	91559	5%	1 minute
Mycobacterium bovis [Tuberculosis]	35743	5%	1 minute
Trichophyton mentagrophytes [Athletes Foot fungus]	9533	5%	1 minute
Coronavirus	VR-1558	5%	1 minute
Hepatitis B	n/a	5%	1 minute
Hepatitis C	n/a	10%	1 minute
Herpes Simplex virus type 1	VR-539	5%	1 minute
Human Immunodeficiency virus [HIV-1] [AIDS]	n/a	10%	1 minute
Influenza virus type A2 [Flu]	VR-544	5%	1 minute
Norovirus	n/a	5%	1 minute
Poliovirus	VR-1562	5%	1 minute
Respiratory Syncytial virus [RSV]	VR-26	5%	1 minute
Rhinovirus	VR-340	5%	1 minute
Rotavirus, Strain WA	n/a	5%	1 minute

Manufactured by:
Rx SAN, LLC
1877 E. 17th Ave.
Columbus, OH 43219

EPA No.: 88494-2-99437
EPA Establishment No.: 52823-OH-1

BATCH NUMBER: